


# Instruction Sheet

Thank you for purchasing the support handle. Please read the following instructions for your safety and to learn of the features of this product.



1. Remove support handle cap by unscrewing the cap in a counter clockwise direction.
2. Insert the two AA-size batteries into the support handle with the positive side of the batteries pointing toward the cap of the support handle.

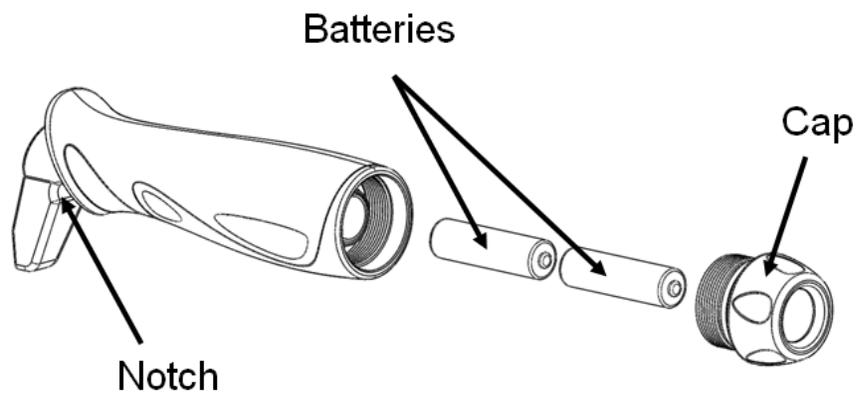

3. Replace the support handle cap by screwing the cap in a clockwise direction.


**NOTE:** Screw the cap in a clockwise direction until the flashlight turns on and then unscrew the cap in a counter clockwise direction until the flashlight turns off (this should only be a few turns). This is how you turn the flashlight on and off. IF YOU DO NOT TURN FLASHLIGHT OFF, THE BATTERIES WILL DRAIN.

4. Install the support handle into your car door latch as shown in the pictures below.

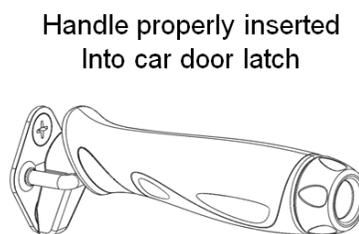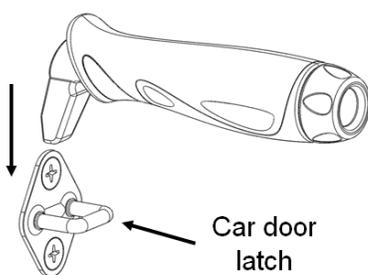

**IMPORTANT:** Do not attempt to insert the support handle into the car door latch in any other direction other than as shown in the above pictures. Failure to do so may cause injury and could possibly damage your car door latch. THE FLASHLIGHT SHOULD POINT TOWARD THE FRONT OF YOU CAR.

Notice the picture below represents the support handle and car door latch in a side view so you can see how the notch of the support handle is inserted into car door latch at the proper depth. This proper engagement is critical for your safety before applying any downward force on the handle.

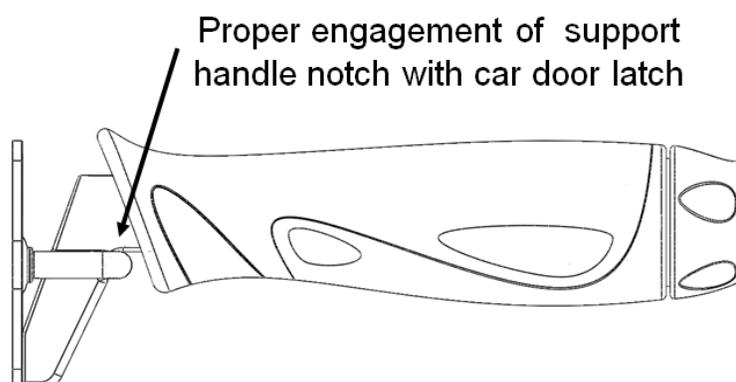

## **WARNING: Please read before using this product.**

1. Serious injury or death can result if the support handle is not properly installed. Standers Inc. is committed to providing safe products and informing the user of ways to prevent accidents.

### **SAFETY PRECAUTIONS:**

- ✓ Make sure the support handle is properly inserted into the car door latch. The support handle must be insert into the car door latch as far as possible before a downward force is exerted.
  - ✓ Additional safety measures should be taken for users that are a high risk. High risk users include those who are not mentally conscious (including those individuals under medication), users who have size or weight conditions, or any condition that impair their ability to use the support handle without assistance.
  - ✓ If you have any concerns or questions regarding these precautions, please contact Standers at:  
1-800-506-9901.
2. If you are suffering from an ailment or injury or are taking medications that affect your balance or ability to stand or sit down, you should first consult with a physician before using this product. If the product is not used properly, serious injury or harm could result.
  3. The product was not intended to carry the full weight of an individual. This product is only intended to provide balance and support while sitting and standing.
  4. The product instructions must be carefully followed to ensure that the product is properly assembled and positioned properly. Failure to follow the instructions could result in serious injury to the user(s) of the product.
  5. Children should not be allowed to hang on or play with the product.
  6. **DISCLAIMER:** Neither Standers, Inc. nor the manufacture is liable for any harm or injury that may result or has resulted from the improper use or assembly of the product.

## **Limited Warranty**

Congratulations on your purchase of your Standers brand product. We hope that it will ease your daily living. Please read the following warranty statement for your protection.

Standers, Inc. warrants to the original purchaser that this product will be free from defects in material and/or workmanship for 1 year from the date of purchase.

During the warranty period, Standers, Inc. will correct any defects in material or workmanship, at no charge. The original purchaser, however, must prepay shipping cost. Any replacement parts will be new and are warranted for the remainder of the original warranty or thirty (30) days from the date of shipment, whichever is longer. The original purchaser must notify Standers, Inc. during the warranty period in writing of the defect in workmanship or material. This warranty is not transferable and covers only normal use. **NO SHIPPING CHARGES ARE REFUNDABLE.**

The warranty does NOT cover the following:

- Damage during shipment other than original shipment to retailer.
- Damage caused by unauthorized attachments, alterations, or modifications.
- Damage caused by the use of the product for purposes other than those for which the product was designed or intended.
- Damage caused by any other abuse, misuse, mishandling or misapplication.

Under no circumstances shall Standers, Inc. be liable for any special, incidental, or consequential damages based upon breach of this limited warranty, breach of contract, or strict liability.

THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THE DESCRIPTION ON THEFACE HEREOF. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED (INCLUDING ALL IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE), ARE HEREBY DISCLAIMED.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal right, and you may also have other rights, which vary from state to state.

To obtain service under this warranty, you can reach Standers Inc. at 1-800-506-9901.



EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands  
Tel: +31 (0)70 345 8570  
Fax: +31 (0)70 346 7299

