


## INSTRUCTIONS FOR PIVOTING WHEEL TO FIXED WHEEL REPLACEMENT:

1. Remove existing Bolts and Nuts in the existing *Pivoting Wheel Assembly* using the included Allen Head Wrench.
2. Reinsert the existing Nuts into the *Fixed Wheel Assembly*.
3. Slide the *Fixed Wheel Assembly* into the walker leg and reinsert the existing Bolts. Tighten the existing Bolts with the included Allen Head Wrench.

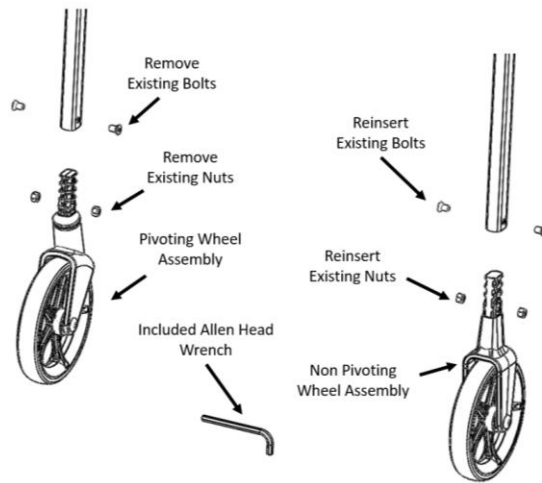

Pivot to Fixed Wheel Replacement V 1.0